

# Face Shield



## 3 Components

1. **Shield:** 13" width x 9.5 " height  
.040" and/or .030" polycarbonate or PTEG supplied with peel-able protector
2. **Foam:** 1" x 1" x 8"  
Pre-applied adhesive
3. **Elastic:** 1" x 18"  
adjustable polyester / rubber non-roll elastic - black or white color

★ Non FDA approved    ★ Non Returnable

## Assembly


Directions as follows:



Kit



Shield

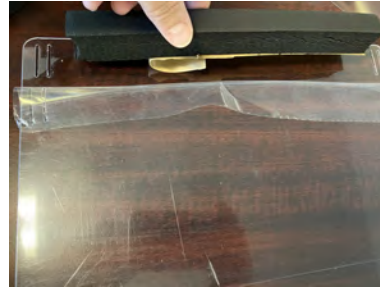

Foam



Elastic (black or white)



Elastic Complete



Finished Face Shield

1. Peel protective liner from shield
2. Attach 1 inch foam centering in between elastic holes
3. Lay shield down foam side up. Working left to right - overhang 4" elastic wrap around shield edge. Go underneath and up through the 2nd slot - then back through the next slot. This should leave a 1" tail Repeat steps on opposite side on shield
4. To adjust shield - slightly bow and tighten by feeding additional elastic and pulling overhang

**Call today for pricing and lead time (866) 284-1541**

Releasor and anyone claiming on Releasor's behalf releases and forever discharges Releasee and its affiliates, successors, officers, employees, representatives, partners, agents and anyone claiming through them (collectively, the "Released Parties"), in their individual and/or corporate capacities from any and all claims, liabilities, obligations, promises, agreements, disputes, demands, damages, causes of action of any nature and kind, known or unknown, which Releasor has or ever had or may in the future have against Releasee or any of the Released Parties arising out of or relating to: InSite Supplied Face Shield ("Claims").